

Caring for Your Ultraclave & Autoclave Sterilizer

MIDMARK®

WEEKLY



WARNING

To prevent burns, allow unit to cool before cleaning.

1. CLEAN EXTERNAL SURFACES

(a) Wipe with a soft dry cloth and occasionally with a damp cloth and mild soap or detergent.

2. CLEAN INTERNAL SURFACES

(a) Drain water from reservoir using drain tube located on front of unit.



EQUIPMENT ALERT

Failure to change water promotes growth of algae in reservoir and may cause sterilizer to malfunction.

(b) Clean trays, door gasket, metal surfaces and inside of chamber with **Speed-Clean Sterilizer Cleaner** and distilled water. Inspect door gasket for damage that could prevent proper sealing.

(c) Refill reservoir with clean, distilled water.

3. DISINFECTING: (Refer to Installation/Operation manual)

MONTHLY





EQUIPMENT ALERT

Failure to flush unit with **SPEED-CLEAN STERILIZER CLEANER**, or use of other sterilizer cleaners may cause some components in unit to fail prematurely.

1. FLUSH SYSTEM

(a) Drain reservoir and fill with clean distilled water. Add 1 oz. of **Speed-Clean Sterilizer Cleaner** to a cool chamber (A).

(b) Run one **POUCHES**  Cycle. Press **STOP**  Button when **Drying** portion of cycle begins.



WARNING

Instruments should not be sterilized while cleaning the sterilizer.


(c) Drain reservoir and refill reservoir with clean, distilled water.

(d) Rinse by running one **UNWRAPPED**  Cycle.

(e) Drain reservoir and allow sterilizer to cool to room temperature.

(f) Remove door gasket, dam gasket and gasket ring. Clean with **Speed-Clean Sterilizer Cleaner** and clean, distilled water. A small stiff bristle brush will aid procedure. After cleaning gaskets, inspect for damage, shrinking, or swelling and replace if necessary. Press gasket and ring into the channel and reinstall dam gasket.

(g) Remove trays, tray rack, and tray plate. Pressing downward on top band of tray rack, pull upward on end of tray plate and slide assembly out of chamber.

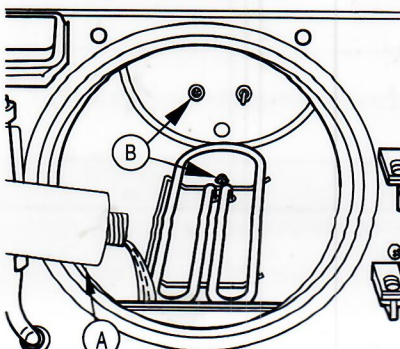

EQUIPMENT ALERT

Do not operate sterilizer without filters in place.

(i) Wipe off trays, tray rack, and tray plate. Reinstall assembly by placing back edge of tray plate in chamber. Pushing downward on top of tray rack, slowly push assembly into chamber.



EQUIPMENT ALERT

Angled end of plate must be toward back of chamber to prevent interference with temperature probe in back of chamber.

(j) Fill the reservoir with clean, distilled water. Sterilizer is now ready for use.

2. PRESSURE RELIEF VALVE CHECK

Refer to installation and Operation Manual for this procedure.

To activate warranty:

Insert CD, then select
Product Registration* from the menu.

(*Midmark recommends product registration
for all Class II devices.)

Remember to ask your dealer for
Speed-Clean Sterilizer Cleaner

- **Français:** Référez-vous à CD-Rom
- **Deutsch:** Siehe CD-Rom
- **Español:** Refiera a CD-Rom

Midmark Corporation
99 Yulet Street
P.O. Box 286
Versailles, Ohio 45380-0286
1-800-MIDMARK
Fax 937-526-5542
midmark.com

Operating Your Ultraclave & Autoclave Sterilizer

PRE PROGRAMMED OPERATION

STEP 1 Select and press the appropriate sterilization preprogrammed button.



{NOTE: Refer to Standard Cycle Parameters (below) to select the proper sterilization program time and temperature.}

STEP 2 Press the START button.



WARNING

Stop button may be depressed at any time to stop or interrupt a cycle. Contents are not sterile if this occurs before the dry cycle begins.

PROGRAMMING

STEP 1 Press button 1 **1** Or 2 **2**.

STEP 2 Press PROGRAM **P** button.

{NOTE: Sterilization temperature can be adjusted from a minimum of 230°F {110°C} to a maximum of 275°F {135°C}.}

The **+** button raises temperature 1°.

The **-** button lowers temperature 1°.

{NOTE: If STOP **⏏** button is pressed anytime during the Programming Mode any settings entered will be aborted and programming will revert back to the original settings.}

STEP 3 Press PROGRAM **P** button.

{NOTE: Sterilization time can be adjusted from a minimum of 3 minutes to a maximum of 90 minutes.}

The **+** button raises time 1 minute.

The **-** button lowers time 1 minute.

STEP 4 Press PROGRAM **P** button.

The **+** button changes venting to FAST.

The **-** button changes venting to SLOW.

STEP 5 Press PROGRAM **P** button.

{NOTE: Drying Time can be adjusted from a minimum of 0 to a maximum of 60 minutes.}

The **+** button raises time 1 minute.

The **-** button lowers time 1 minute.

STEP 6 Press PROGRAM **P** button.

The display shows the new programmed settings for the button that was programmed, 1 **1** Or 2 **2**.

{NOTE: The programmed settings entered are retained under that program button (1 or 2). If power is interrupted or the unit is unplugged, the settings will be retained.}

STANDARD CYCLE PARAMETERS



EQUIPMENT ALERT

Using an incorrect sterilization program could result in non-sterile contents and may damage instruments. Consult instrument manufacturer for specific sterilization instructions.



EQUIPMENT ALERT

Trays must be used at all times when operating this sterilizer, serious equipment damage could result.

	Unwrapped	270°F (132°C) 27.1 psi (186 kPa) Sterilize for 3 minutes Dry for 30 minutes	<ul style="list-style-type: none"> Instruments loose on a tray. Open glass or metal canisters. Tubing not used in surgical procedures. Loose items manufacturers recommend for exposure at 270°F (132°C). <p>The sterility of unwrapped items is compromised on exposure to a non-sterile environment.</p>
	Pouches	270°F (132°C) 27.1 psi (186 kPa) Sterilize for 5 minutes Dry for 30 minutes	<ul style="list-style-type: none"> Pouched or loosely wrapped instruments. Wrapped trays of loose instruments. Tubing not used in surgical procedures. Wrapped items manufacturers recommend for exposure at 270°F (132°C).
	Packs	250°F (121°C) 15 psi (104 kPa) Sterilize for 30 minutes Dry for 30 minutes	<ul style="list-style-type: none"> Textiles and surgical packs wrapped for sterilization. Items, except liquids, manufacturers recommend for exposure at 250°F (121°C).
	Handpieces	270°F (132°C) 27.1 psi (186 kPa) Sterilize for 6 minutes Dry for 30 minutes	<ul style="list-style-type: none"> Dental handpieces. Loose or wrapped dental handpieces manufacturers recommend for exposure at 270°F (132°C).
1	Program	Programmable User Defined 230°F (110°C) to 275°F (135°C)	<ul style="list-style-type: none"> Items appropriate for user defined parameters
2	Program	6 psi (41 kPa) to 31 psi (214kPa) Sterilize for 3 min to 90 min Dry for 0 to 60 min	<p>CAUTION Temperatures below 250°F (121°C) should not be used for disinfection unless otherwise recommended by the manufacturer.</p>