Documents
KaVo QUATTROcare® 2104/2104 A

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KaVo. Dental Excellence.

Vertrieb/distribution:

KaVo Dental GmbH

Bahnhofstraße 20 • D-88447 Warthausen

Tel.: 0 73 51 / 56-0 • Fax: 0 73 51 / 1 22 14

Hersteller/manufacturer:

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Bismarckring 39

D-88400 Biberach

DE

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GB

A	Instructions for use	4
A 1	Important notes.....	4
A 1.1	Operating environment.....	8
A 1.2	Introduction.....	10
A 1.3	Technical customer service.....	12
A 2	Device description.....	14
A 3	Operation/handling.....	16
A 3.1	Attaching the instruments.....	16
A 3.2	Care sequence.....	16
A 3.3	Removing the instruments.....	18
A4	Rectifying faults.....	20
B	Installation instructions	22
B 1	Packing list.....	22
B 2	Commissioning.....	24
B 2.1	Inserting the spray can.....	26
B 2.2	Removing the spray can.....	26
B 2.3	Disposal of the spray can.....	26
B 2.4	Care couplings.....	28
B 2.5	Air purging time.....	28
B 3	Accessories.....	30
E	Maintenance	32
F	Technical data	32
	EC-Conformity Statement.....	34
K	Guarantee	36

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KaVo China
Service and Sales office
Block 2 Huiwan Overseas-
Chinese Apartments
Area 3 of Huianli
Chaoyang District # 12-03
Unit 2
Beijing 100101
Tel. ++ 86 / 10 64 / 91 65 87
Fax ++ 86 / 10 64 / 91 65 87

KaVo Polska sp.z.o.o.
ul. Pomorska 251 P 405
92-214 Lodz
Polska
Tel. ++ 48 / 42 / 6 75 75 36
Fax ++ 48 / 42 / 6 75 75 36

KaVo Osaka
Yotsubashi Okawa Building
11 Fl. 1-6-23, Shinmachi
Nishi-Ku
Osaka 550-0013
Japan
Tel. ++ 81 / 6 / 65 38 58 50
Fax ++ 81 / 6 / 65 38 58 55
e-mail: info@kavo.co.jp

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A 1 Important notes

Please read the instructions for use carefully before you commission the QUATTROcare, and keep the handbook for later use. In order that the device functions in the envisaged manner, the instructions for operation, maintenance and the installation of spare parts must be followed exactly.

The trademarks listed are the property of the relevant holder.

The QUATTROcare is:

- intended for internal cleaning and maintenance of straight and contra-angle handpieces and turbines in the dental medical sector.
- a technical aid according to the law relating to technical aids and to be assigned to the law relating to device safety.

According to these regulations, the QUATTROcare must only be used by the expert user for the application described, taking into account:

- the applicable health and safety regulations,
- the applicable accident prevention measures
- and these instructions for use.

According to these regulations, it is the duty of the user

- to only use fault-free aids, pay attention to the correct intended use and protect oneself and others from hazards.

Electromagnetic compatibility

In accordance with legal stipulations governing electromagnetic compatibility (EMC -DIN EN 6061-1-2, of October 2002), we must point out that:

- Electrical medical devices are subject to special EMC safety measures and, as a result, the KaVo Assembly instructions must be closely adhered to.
- Portable and mobile high-frequency electronic communications equipment may interfere with electrical medical devices.
- Further information about technical electromagnetic compatibility requirements can be provided upon request.

Please note that the EC Directive on waste electrical and electronic equipment applies to this product. Within Europe therefore, this product must undergo special disposal. For more detailed information about this, please contact KaVo or your specialist dental supplier.

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Disposal of wastes and residues from the device and from the accessories at the end of the period of use

The wastes produced are to be sent for recycling of the materials or for disposal in a manner which poses no threat to humans or the environment; compliance with National regulations is essential.

Please read before commissioning

The QUATTROcare has been developed and produced in accordance with the latest quality and safety standards.

However, some precautions should be taken during installation and operation.

- The QUATTROcare is not intended to be used in explosion hazard areas.
- The operator of the QUATTROcare is requested to have changes or overhaul of the electrical equipment carried out by ones dental depot, in the manufacturer's factory or in a KaVo subsidiary.
- When the practice is vacated, the QUATTROcare should be isolated from the mains.
- Protect QUATTROcare with inserted spray can from being heated above 50° C.
- The information on the type label of the power supply unit (voltage, frequency) must correspond to the mains supply available.
- In the event of changes made by third parties, the warranty is nullified. It is recommended that only KaVo original spare parts be used for operation or for repair since these have been tested extensively with regard to their safety, function and specific suitability. KaVo therefore assumes responsibility for these parts too.
- Please observe all the warning notes on the device and in the instructions for use.
- The device must only be supplied with the voltage which is stated in the instructions for use or displayed on the device.
- Users must not attempt to maintain the device themselves beyond the items in the instructions for use.

Optional for QUATTROcare 2104 A option:

- When the practice is vacated, the QUATTROcare must be isolated from the electrical mains and from the compressed air supply.

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A 1.1 Operating environment**Temperature and humidity**

Please take care that the QUATTROcare is neither exposed to direct sunlight nor placed in the vicinity of a heat source. The operating temperature should be between 15°C and 40°C, the atmospheric humidity should be between 25% and 90%.

Spacing

The ventilation slots and openings on the QUATTROcare must not be covered or closed. In general, therefore, a spacing of 50 mm should be maintained on all sides between the device and adjacent walls or cabinet surfaces.

Ventilation

The QUATTROcare should be operated in a clean environment, such as that found in a laboratory or dental practice.

Standing surface

The QUATTROcare should be set up on a flat, horizontal surface which is not sensitive to water or oil. The QUATTROcare must never be set up or operated on an inclined surface.

Suppression of interference

According to an official test, the QUATTROcare meets the requirement of Directive 89/336/EEC concerning electromagnetic compatibility. Even though the QUATTROcare itself does not emit any interfering radiation, it is possible for other devices (e.g. ultrasonic cleaners) to cause interference with the QUATTROcare. Therefore, keep the QUATTROcare away from possible interference sources.

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Electrical safety

The QUATTROcare must only be connected to a properly earthed and safeguarded power source.

The power source and QUATTROcare must have the same nominal voltage. The nominal voltage is indicated on the rating plate on the rear panel of the device and on the power input window.

If you need an extension cable, use only cables which are approved for the nominal voltage indicated on the type label of the QUATTROcare and have a protective earth conductor.

A 1.2 Introduction**Significance of the pictograms**

 *A situation which, if the instruction is not complied with, can lead to a hazard, damage to material or to an operational fault.*

 *Important information and instructions for the operator and engineer.*

Type label

- ① Device type
- ② Serial number
- ③ Material number
- ④ Technical data
- ⑤ Continuous operation with intermittent loading.
- ⑥ VDE trade mark
- ⑦ CE mark
- ⑧ CSA mark
- ⑨ Important: Note the accompanying documents and safety information.
- ⑩ This arrow indicates the location of the mains input fuses.
- ⑪ For information about proper disposal, see A1

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A 1.3 Technical customer service

Technical care for KaVo products will be carried out in the first instance by the dental depot.

Engineers at the dental depot and KaVo Customer Service engineers participate frequently in training sessions and special training courses at the factory.

In order to ensure that KaVo products are always ready for use and maintain their value, a service should be carried out regularly.

Please contact KaVo Biberach, a KaVo subsidiary (see page 3) or your KaVo agent with any inquiries.

In the event of queries, the type designation and serial number must always be specified.

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A 3 Operation/handling

To open the device, grip the recessed handles on the left and right of the front flap and pull it towards you and down.

 *Do not place any articles on or in the front flap.*

A 3.1 Attaching the instruments

- Attach straight handpiece and contra angle shanks to the INTRAmatic care coupling, latching them in securely.
- In the case of LUX instruments, the position of the fixing lug should be noted.
- Attach contra-angle turbine to the Multiflex care coupling and snap in.
- Insert heads into the INTRA head care coupling (special accessory), push back locking sleeve ① and snap in.

 *When installing the care coupling for INTRA heads, it is essential to push the switch sleeve ② onto the associated flap in order to ensure safe care.*

Push front flap upwards to close it.

 *It is not necessary to mount all adapters, nor in any order. Adapter 1 is on the left, as well as LED display 1.*

A 3.2 Care sequence

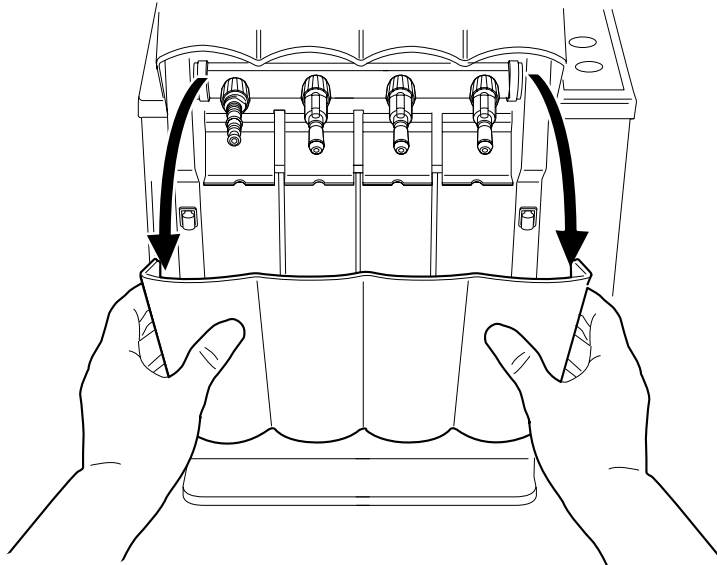
When the main switch is on, the LED ④ lights up. Operate the start push-button ⑥, and the automatic care sequence begins.

 *The automatic care sequence lasts approximately 20 seconds per instrument.*

After the end of the care sequence, the LEDs ⑤ light up to indicate whether the care couplings have instruments attached, and an acoustic signal is heard. The LEDs ⑤ remain lit up until the front flap is opened. The care sequence has been completed.

 *If the LED ③ lights up, the spray can is empty or not correctly inserted.*

After the front flap has been opened, the LEDs ⑤ go out.

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A 3.3 Removing the instruments

i Prior to removing instruments use a cloth to remove excess oil from the exterior of the handpieces.

■ When removing straight handpieces and contra angle shanks, operate the latch with the thumb and pull the instrument off in an axial direction.

■ Pull the turbine handpiece off in an axial direction to remove it.

■ To remove INTRA head, push back the locking sleeve and remove head in an axial direction.

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A 4 Rectifying faults

P Red LED ① flashes and a sound is emitted

C Spray can not correctly inserted.

S Insert spray can correctly (see Section B 2).

C Spray can is empty.

S Insert new spray can.

P Green LEDs ② flash and a sound is emitted

C Front door was opened before cycle was complete.

S Wait until the care cycle has been completed and the signal tone sounds. Repeat the care sequence.

C Instrument has fallen off the adapter during the care cycle.

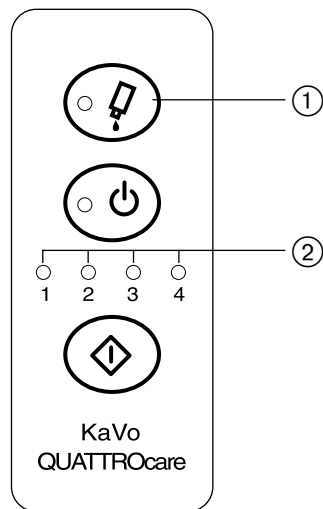
S Attach the instrument correctly. Repeat the care sequence.

QUATTROcare 2104 A

P Oil emerges from the instrument after care.

C Air purging time is set too short

S Set air purging time longer (see Section B 2.5).

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B 2 Commissioning**Commissioning the QUATTROcare 2104**

Check the mains voltage adjustment ①. If the mains voltage is wrong, press in the catch of insert ① and remove. The insert contains 2 T500 H250V fuses. Pull out grey insert, turn and reinsert. The desired voltage must be legible in the inspection window of insert ①. Push insert with fuses into the device.

Plug the mains plug into socket ③. Switch on main switch ②.

Commissioning the QUATTROcare 2104 A

Check the mains voltage adjustment ①. If the mains voltage is wrong, press in the catch of insert ① and remove. The insert contains 2 T500 H250V fuses. Pull out grey insert, turn and reinsert. The desired voltage must be legible in the inspection window of insert ①. Push insert with fuses into the device.

Plug the mains plug into socket ③. Plug the compressed-air hose onto coupling ⑤ and secure.

The compressed air must indicate 4 - 4.5 bar on the manometer ④.

If the indication is different, pull out the handle on the pressure regulator ⑥ and rotate to right or left until 4 - 4.5 bar is indicated.

Press in the handle on the pressure regulator to lock the adjustment.

 *The compressed-air hose must not be kinked.*

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B 2.1 Inserting the spray can

Open the side door. Insert the spray can obliquely into the holder and press it upwards until it latches in audibly.

A magnet holds the spray can firmly.

Check for correct and firm seating in the vertical position.

If the spray can is not snapped in correctly, a beep tone sounds when the device is started, and LED ① lights up.

This warning signal is also used to indicate an empty spray can.

 Use only original KaVo QUATTROcare spray cans.

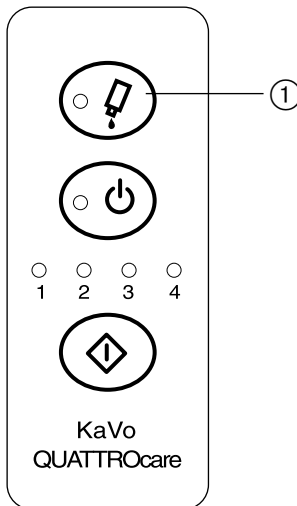
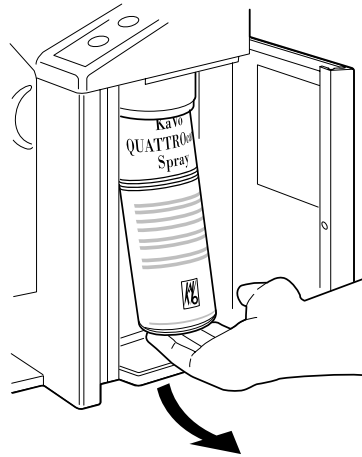
B 2.2 Removing the spray can

Hold the spray can underneath at the bottom and remove it, pulling obliquely.

 If long intervals between uses are envisaged, it is advisable to remove the spray can from the device.

B 2.3 Disposal of the spray can

Operate the spray can until it is completely empty. No special measures are then necessary for the empty spray can. Waste code according to the European waste catalogue: 15 01 04 - Metals, packaging, emptied.

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B 2.4 Care couplings

When supplied, the QUATTROcare is equipped with:
3 off care couplings for INTRAmatic
1 off care coupling for Multiflex

Changing the fittings

Open the union nut by turning clockwise as indicated by the arrow and remove care coupling. Attach selected KaVo care couplings and screw tight by turning the union nut counterclockwise, opposite to the direction of the arrow.

i *Couplings with 4-hole standard connection for products from other manufacturers can also be attached.*

B 2.5 Air purging time

In the case of QUATTROcare 2104 A, the air purging time for straight and contra-angle handpieces and turbines can be changed. Keep start button ⑤ depressed and switch on main switch. The LED 1 lights up. If start button ⑤ remains depressed, the LEDs 2, 3 and 4 light up in succession. After the desired air purging time (LED 1, 2, 3 or 4), release the button. A signal is heard.

The air purging time is as follows:

LED 1 - 20 seconds
LED 2 - 25 seconds
LED 3 - 30 seconds
LED 4 - 40 seconds

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
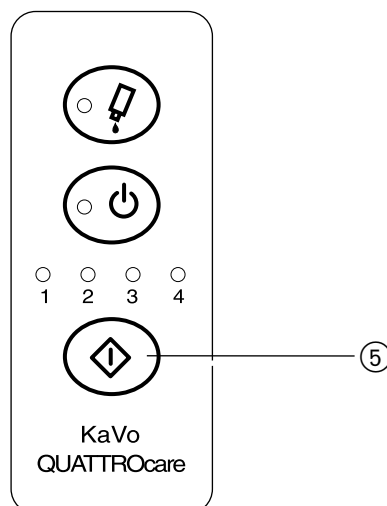
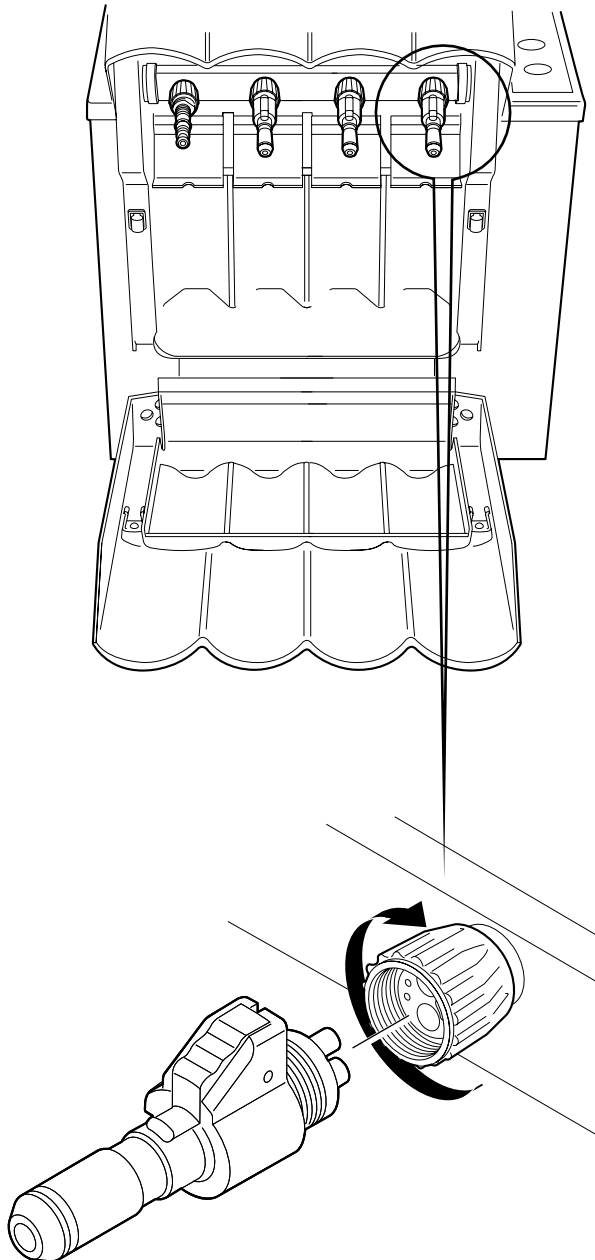
B 3 Accessories

Obtainable via the dental medical specialist trade

Description	Mat.-No.
① O ring, INTRAmatic	0.200.6089
② Care coupling, INTRAmatic	0.411.7981
③ Seal, care coupling	0.553.1872
④ Care coupling, Multiflex	0.411.7991
⑤ O ring, Multiflex	0.200.6098
⑥ O ring, Multiflex	0.200.6177
⑦ Head care couplings incl. switch sleeve	0.411.7941 0.411.5532
⑧ Maintenance adaptor SIRONA	1.000.1003
⑨ Care coupling SIRONA T1 turbine	1.000.7156
⑩ Care coupling YOSHIDA turbine lux	1.000.7403
⑪ Care coupling YOSHIDA turbine	1.000.7585
⑫ Care coupling MIDWEST Low Speed	1.000.8459
⑬ Care coupling NSK turbine	1.000.8786
⑭ Care coupling OSADA turbine	1.000.8858
⑮ QUATTROcare spray 2108A Pack of 6	0.411.7720
⑯ Sprayhead clamping ring maintenance for QUATTROcare spray	1.003.1253

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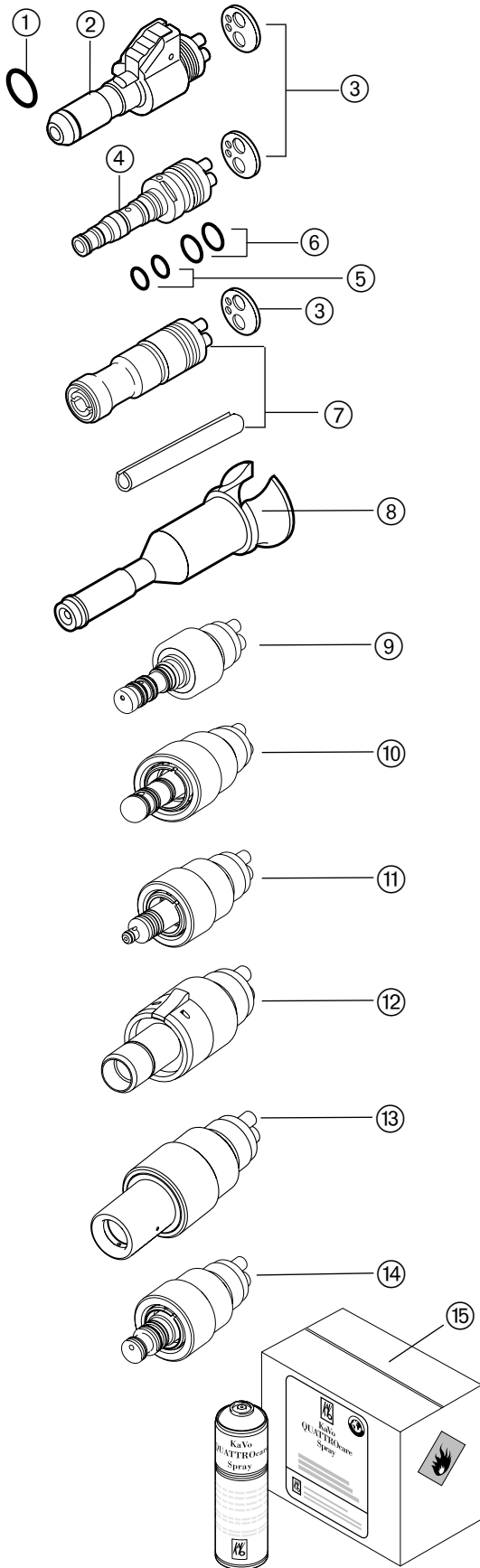E 1 Maintenance**Cleaning the device**

Clean all the outer surfaces of the QUATTROcare with a soft cloth and a mild cleaning solution.

Do not use any solvents or aggressive chemicals.

 When cleaning the device, take care that no liquid gets into the interior of the device, since this can lead to malfunction.

Wipe off or clean the inner walls regularly. Remove the oil trap tray ① and clean it. It may be washed in the equipment washing machine or in the thermal disinfecting machine.

Temperature: max 95° C

Time: max 60 min

When inserting the oil trap tray ①, ensure it is correctly snapped in.

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F 1 Technical Data**Device dimensions**


Width	350 mm
Depth	280 mm
Height	380 mm

Weight	10 kg
--------	-------

Power consumption	230 V / 50 Hz / 20 W
-------------------	----------------------

Optional for QUATTROcare 2104 A:

Compressed-air connection	4 - 6 bar
Air consumption	approx. 50 NL/min

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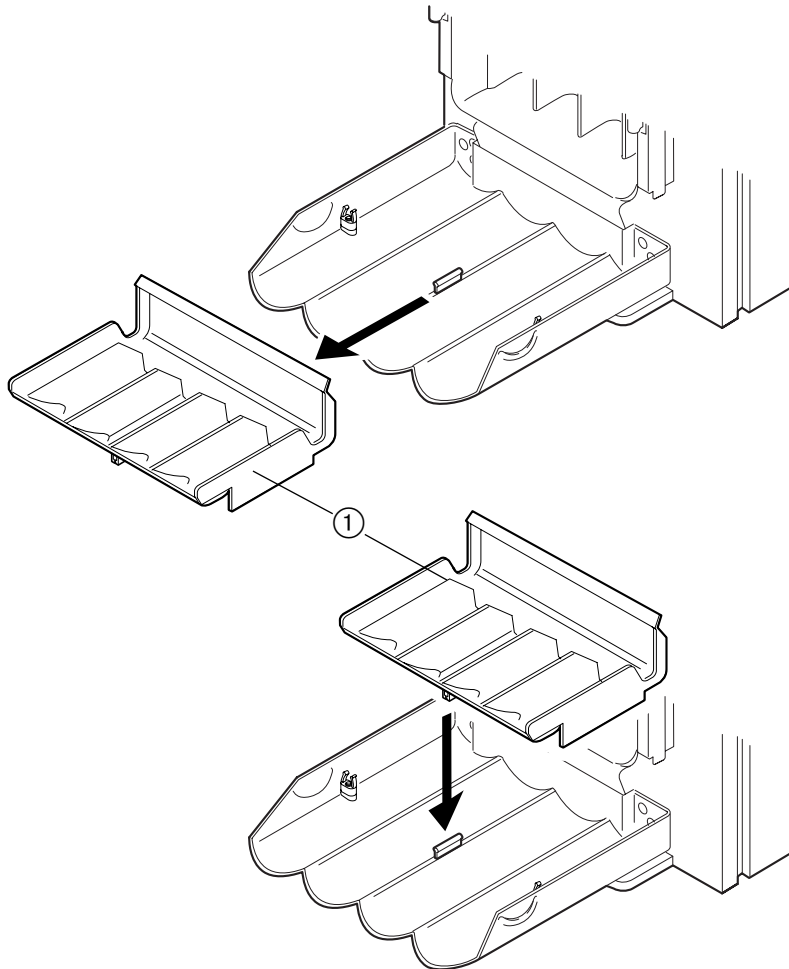K 1 Guarantee conditions

KaVo Dental GmbH undertakes to provide the final customer of this product with a guarantee of satisfactory function, freedom from faults in both and manufacturing process for the duration of 12 months from the date of invoice, under the following conditions:

In the case of justifiable complaints, KaVo Dental GmbH will service the unit by means of overhaul or the supply of spare parts at no charge. All other claims, of whatever type, in particular for compensation, are excluded.

KaVo Dental GmbH accepts no responsibility for defects and their consequences which have arisen or could have arisen as a result of natural wear, improper handling, improper cleaning, maintenance or care, non-compliance with the operating instructions and chemical or electrical influences which are unusual or are not admissible in accordance with the KaVo Instructions for Use and in accordance with other manufacturer's instructions. The guarantee generally does not extend to lamps, optical conductors made of glass and glass fibres, glassware, rubber parts and to the colour-fastness of plastic parts.

Any responsibility is excluded if defects or their consequences are based on the fact that the customer or third parties not authorized by KaVo, have undertaken actions or changes to the product.

 *Guarantee claims can only be validated when a proof of purchase in the form of a copy of an invoice or delivery note is submitted with the product. The dealer, the sale date, the device number or type and the fabrication number or serial number must be clearly visible from this proof of purchase.*

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